


Rapid Anti-HCV Test

(Whole Blood, serum or plasma)



As shown above, prepare all items for testing. (Alcohol Swab, Lancet, Timer and Gloves are not provided in the test kit, but they must be available.)

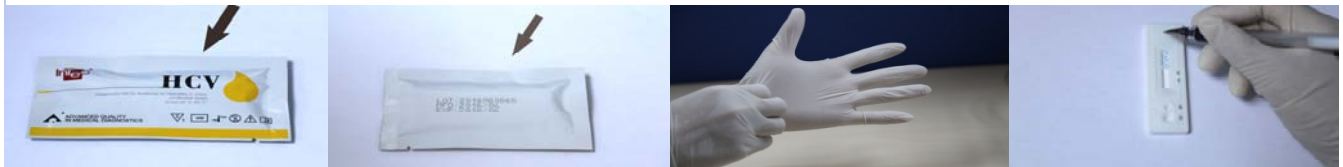


Confirm product name and check that the packet is still

Check the **expiry date** on the test packet. (If expired, do not use).

Put on gloves. (Use new gloves for each client).

Write a Client Identifier on the test.

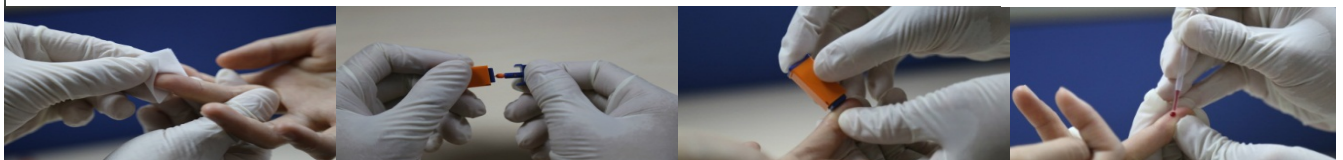

Wipe the client's finger with the alcohol swab. (Finger must be dry before pricking).

Twist the top of the lancet clockwise and remove it.

Press Lancet firmly on the finger and lancet will trigger. **Wipe away the first drop of blood.** Discard the lancet into the sharps bin immediately.

Gently squeeze the bulb of the pipette and touch the tip to a drop of blood. Release the bulb to draw up the blood.



Add 1 drop(10µl) of blood into the "S" Well.

Add 2 drop of diluent (2*50µl) into the "D" Well.

Read result at 15 minutes. Don't interpret after 20 minutes.

INTERPRETATION

